
**QFT™**  
QuantiFERON-TB Gold

## Blood Collection Tubes

Single Patient Pack  
Cat.No: 0597-0201

**CONTENTS**


3x 1mL QFT™ blood collection tubes.  
1x 21G 1½" needle.  
1x QUICKSHIELD™ Safety Tube Holder.  
1x Package Insert.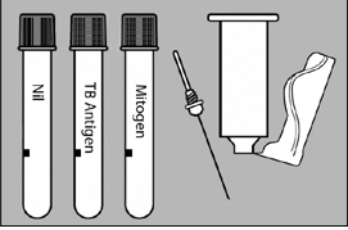

For use with QFT™ ELISA kit (Cat.No: 0594-0201).

**1**

### BLOOD COLLECTION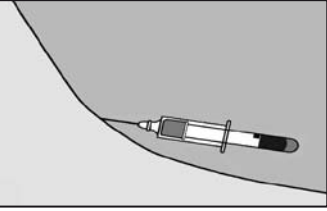


**WARNING:**

- Standard blood handling precautions apply.
- Tubes may be stored and must be at room temperature (17°C–27°C) at the time of use.


Collect blood by venepuncture.  
*Vacuum fills to black mark (1mL).  
Blood flow may be slow.  
Hold tube on needle for 2–3 seconds after flow ceases.  
Repeat tube if not close to mark.*

**Technical Tip**  
Butterfly needles—prime the tubing with a normal tube (not supplied) before filling QFT™ tubes.

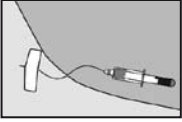**2**

### BLOOD COLLECTION

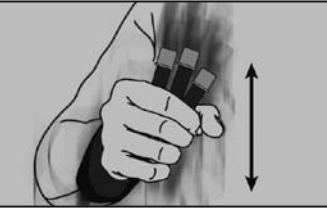
Immediately after filling, **SHAKE** tubes **vigorously** for 5 seconds (10X).  
*Ensure that entire inner surface of each tube has been coated with blood.*

**Proper shaking will lead to frothing of blood. This does not affect the results, and should be observed for correct performance of the test.**  
*Label tubes appropriately.*

**3**

### SHIPPING / INCUBATION

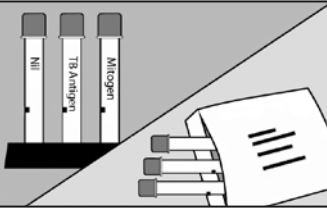**OPTION 1**  
**Incubate at Laboratory**

Ship tubes to laboratory at 17°C–27°C. Blood must be incubated at 37°C as soon as possible (and within 16 hours of collection).  
*Laboratory staff must re-shake tubes immediately prior to 37°C incubation.*

**Technical Tip**  
Label tubes as “Not Incubated”.

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
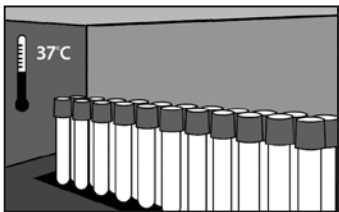**SHIPPING / INCUBATION****OPTION 2  
Incubate at  
Collection site****5**


Blood must be incubated as soon as possible (and within 16 hrs of collection). Incubate tubes **upright** at 37°C for 16–24 hours.

*Portable incubators are available from Cellestis.*

**Technical Tip**

Humidity/CO<sub>2</sub> not required.

If tubes are not incubated at 37°C soon after collection, re-shake tubes immediately prior to incubation.

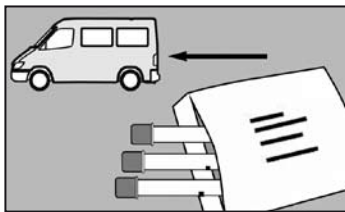
**SHIPPING / INCUBATION****OPTION 2  
(continued)****6**

Ship **incubated** tubes to testing laboratory (within 3 days, if not centrifuged).

*Maintain tubes at 2°C–27°C.*

**Technical Tip**

Label tubes as “Incubated”.

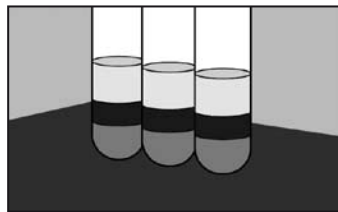**CENTRIFUGATION**

Centrifuge tubes prior to sampling.

*Spin at 2000–3000g for 15 minutes.*

Plasma in **centrifuged** tubes is stable for 4 weeks when stored at 2°C–8°C.

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Cellestis GmbH
Cellestis Limited