

Technical Note

Residues of post coat solution in QuantiFERON® microplates

The application of a post coat solution is the final part of the manufacturing process of the QuantiFERON ELISA microplates.

This solution is clear when first used, and becomes a light brown to yellowish colour with time after drying.

All microplates undergo a careful inspection prior to bagging them. However, when some of the post coat solution gets on the sides of the plate wells it can be difficult to see until well after it is dried.

Corrective measures and improved inspection steps have reduced the incidence of this unusual occurrence with the product, and further improvements are being investigated.

Testing has shown such residues of post coat solution do not have any adverse effect on assay performance.



For more information, please contact the Cellestis office nearest you or visit www.cellestis.com.

North America / South America
Cellestis Inc.
Email: customer.service@cellestis.com
Fax: +1 661 775 7479
Tel: +1 661 775 7480 (outside USA)
Toll free: 800 519 4627 (USA only)

Europe / Middle East / Africa
Cellestis GmbH
Email: europe@cellestis.com
Fax: +49 6151 428 59 110
Tel: +49 6151 428 59 0

Asia / Oceania
Cellestis International
Email: quantiferon@cellestis.com
Fax: +61 3 9571 3544
Tel: +61 3 9571 3500